ESPAÑOL

Aviso

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ENGLISH

Notice

Notice to the user and/or patient: any serious incident that has occurred in relation to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

Please read these instructions carefully so that you are familiar with the Mezquita walker before using it:

1. Remove the walker from its packaging and place the legs on the floor. Unfold it by opening the side frames until you hear the safety click.
2. Adjust the height to the correct level. To do this, press the pin with your finger, extend the legs to the desired height and fix it in place.
3. Check that the armrest and handle are tight and be careful not to hurt yourself if they are loose.
4. Make sure that the rubber non-slip pads on the legs are tight and secure. If they are worn, replace them immediately. Check them especially if you use the walker on slopes.

Using the walker

First, place the walker one step in front of you and make sure that the legs are at the same height. Hold on to the top of the walker by the handles for support and walk towards it. To fold, press the locking pin on the upper front tube and fold the side frames over the front frame.

Directions

- It is designed to assist walking for the elderly, disabled and people who have difficulty walking.
- Weight capacity of 100 kgs.
- Ensure that the walker is adjusted to the correct height and that the locking springs are fixed and protrude through the holes in all legs and the top.
- Fasten all screws securely and tightly.
- Before using the walker, check that the side frames are fully extended and locked.
- Check that the rubber feet and handle are in good condition before use.

- Remember that this product should only be used on flat, smooth surfaces.

Tips and precautions

- Do not apply force on one side only or the walker will tip over.
- Do not go up or down stairs or steps with the walker.
- Do not adjust the walker during use. Assembly and disassembly of the walker should be done only when the walker is not in use.
- Do not walk on escalators with a walker.
- Do not change the direction of the walker suddenly when using the walker on a slope.
- It should be used on level streets. Do not use on streets with potholes.
- Avoid bumping the walker against obstacles.
- Do not walk with an uphill walker with an incline of more than 7 degrees. Do not walk with a walker downhill with a slope of more than 15 degrees. When walking uphill or downhill, the lateral incline or decline of the slope must not exceed 3.5 degrees.
- Do not hold the walker by the front bar or use the front bar to sit on.
- Avoid exposing the product to sunlight.

Cleaning and Maintenance

- Clean the rollator regularly with a clean, dry cloth.
- Store and maintain the rollator in a cool, dry place and avoid excessive moisture.

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GUANGDONG DAYANG MEDICAL TECHNOLOGY CO., LTD
SUNGO EUROPE B.V.
Imported by/Importado por: Grupo R. Queralto S.A.