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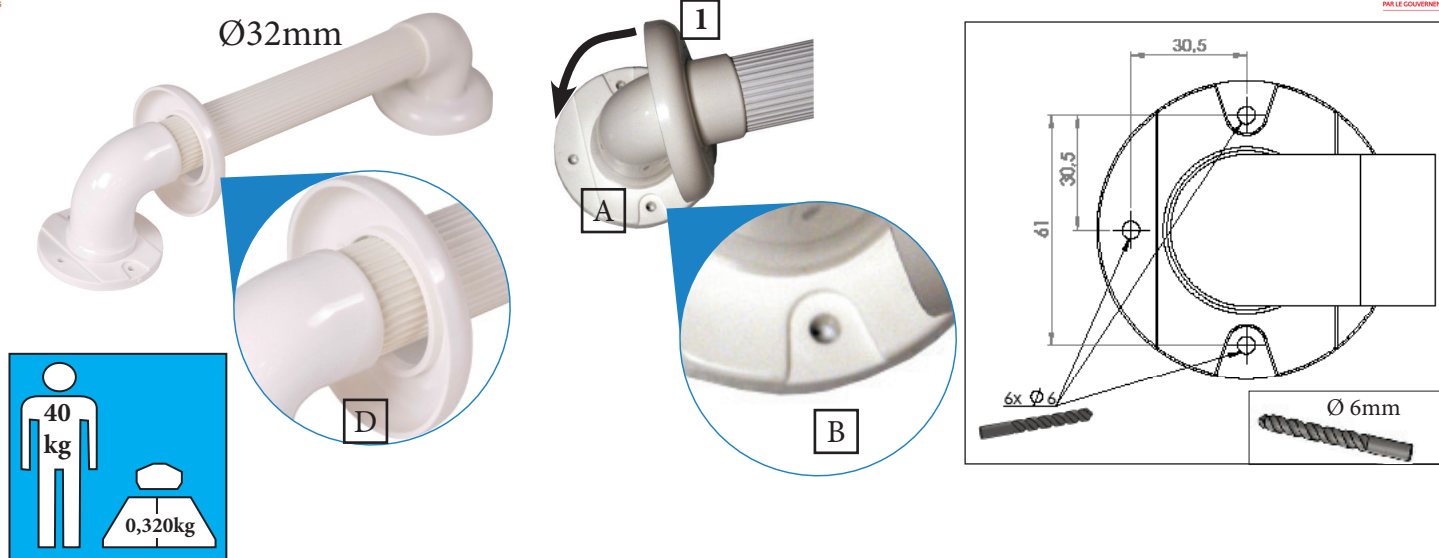
Norm: EN-ISO 111 99-1

EN- Please read these safety instructions carefully before use, and keep them for future questions. If somebody else should use this product, ask him to read carefully these instructions.

1. Safety indication

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Grab bar may be fitted to most types of solid walls. Make sure that you are using suitable plugs and screws. In case of doubt, ask a technician to install the product.
- ♥ Grab bar is to be used as a balance assist only. Not for body weight leverage. Don't submit the grab bar to pressure over 40kg.
- ♥ Always make sure the grab bar is securely attached, before each use. In case of doubt, ask a technician to reinstall the grab bar.
- ♥ Do not store or mount outdoor, the grab bar must be used at home
- ♥ Use only as a safety grab bar and in proper way : don't stand up on it.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ In case you didn't use your gab bar for a long time, let it checked by the technical staff, before use.
- ♥ DO NOT hang anything on the grab bar.

- ♥ Check the grab bar before use, in case of doubt ask your physician or provider.

2. Installation

Beware ! The grab rail should be fitted by competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user.

The “PROFILO” ABS fluted grab bars are critical for many people with mobility impairments. Our grab bars are suitable for use in a variety of areas around the home. Their fluted design provides extra grip and makes them ideal for wet areas such as shower or bathroom.

Diameter of rails 32mm. Distance from wall 40mm. Flanges cover the screws for an esthetic purpose. Maximum traction force on the garb bar is 40kg.

Tools required : a screw driver, a drill, screws and fixing plugs.

Only specialists can mount the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

1- Remove the grab bar from packaging and check for craks and damages. If you see any, or suspect a fault, do not assemble the product; contact your supplier immediately.

2- Mark the holes [B] on the wall using the mounting plate [A]. Drill [C], insert plugs and screw the grab bar on the wall.

3- Install the flanges [D] by clipping them on the mounting plates (1). They will hide the screws.

4- Check the stability and sturdiness before use

3. Cleaning

The grab bar has to be cleaned regularly with a mild cleaning agent and a soft cloth.

4. Materials

The product is made out of plastic (ABS), it won't rust or corrode.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

Ref 573 030 Length : 320 mm

Ref 573 040 Length : 400 mm

Ref 573 045 Length : 450 mm

Ref 573 060 Length : 600 mm

Diameter of pipes : 32 mm

Distance to the wall : 40 mm

Maximum supported weight: 40kg.

This product has a 1 year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible.

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